# SPECTRE™ guidewire

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### Instructions For Use

#### USA CAUTION

Federal law restricts this device to sale by or on the order of a physician.

#### DEVICE DESCRIPTION

The Spectre guidewire is a nitinol and stainless steel core guidewire with a 0.014" diameter and straight shapeable tip. It is available in 200cm and 300cm lengths. The distal 25cm of the guidewire has a spring coil, of which the distal 3cm is platinum and visible under fluoroscopic methods. The distal 42cm of the guidewire has a hydrophilic coating and the proximal portion has a PTFE coating.

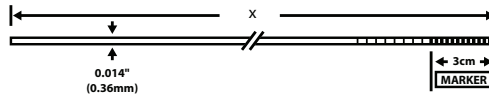
Model 7386 has a modified proximal end to allow for guidewire extension.

The Spectre guidewire has been sterilized with ethylene oxide.

#### SPECIFICATIONS

Model	7386	7387
Length (x)	200cm	300cm
Shaft outer diameter	0.014"	
Tip Outer diameter	0.014"	
Tip Load	0.9g	

#### INDICATIONS

The Spectre guidewire is intended for use in percutaneous procedures to introduce and position catheters and other interventional devices within the coronary and peripheral vasculature.

#### CONTRAINDICATIONS

The Spectre guidewire is contraindicated for use in the neurovasculature.

#### WARNINGS

The Spectre guidewire is provided sterile for single use only. Reuse of a single-use device creates a potential risk of patient or user infections and may compromise device functionality, which may lead to illness or serious patient injury.

Never advance or withdraw an intravascular device against resistance until the cause of the resistance is determined by fluoroscopy. Movement of the guidewire against resistance may result in separation of the guidewire tip, damage to the guidewire, or vessel injury.

#### PRECAUTIONS

Do not use the Spectre guidewire if the packaging is damaged. A damaged package could indicate a breach of sterility or device damage.

Inspect the Spectre guidewire prior to use for any kinks. Do not use a damaged guidewire. Vessel damage and/or inability to advance or withdraw the guidewire may occur.

Exercise care in handling the Spectre guidewire during a procedure to reduce the possibility of accidental breakage, or kinking.

When the guidewire is in the body, it should be manipulated only under fluoroscopy. Do not attempt to move the guidewire without observing the resultant tip response, as guidewire damage or vessel injury may occur.

When shaping the guidewire tip, do not use an object with a sharp edge as this may damage the guidewire.

Always secure the guidewire prior to advancing or withdrawing an interventional device over it. Failure to do so may result in loss of guidewire position and/or vessel injury.

Before inserting the guidewire into an interventional device, always flush the guidewire with saline to ensure the hydrophilic coating is thoroughly wet. Failure to do so may inhibit wire movement within the interventional device.

**For 200cm length only:** Do not attach a torque device to the modified proximal end intended for extension as it may damage the guidewire, preventing the ability to attach to the DOC® Guide Wire Extension.

#### ADVERSE EFFECTS

Potential adverse effects that may be associated with the Spectre guidewire include, but are not limited to, the following:

- Cardiac tamponade
- Embolism
- Hematoma
- Infection
- Inflammatory response
- Thrombosis

- Vessel dissection
- Vessel perforation
- Vessel spasm

#### CLINICAL PROCEDURE

The Spectre guidewire should be used by physicians trained on the procedures for which the device is intended. The techniques and procedures described do not represent ALL medically acceptable protocols, nor are they intended as a substitute for the physician's experience and judgment in treating any specific patient. All available data, including the patient's signs and symptoms and other diagnostic test results, should be considered before determining a specific treatment plan.

#### Package contains:

- 1x Spectre guidewire

#### Other items required but not provided:

- Guide catheter
- Hemostasis valve
- Sterile heparinized saline (for flushing)
- Guidewire introducer (optional)
- Torque device (optional)

#### PREPARATIONS FOR USE

1. Prior to use, carefully inspect the guidewire packaging and components for damage.
2. Utilizing sterile technique, remove the guidewire from its packaging and transfer to the sterile field.
3. Thoroughly flush the guidewire contained within the hoop with heparinized saline to activate the hydrophilic coating.
4. The guidewire tip may be carefully shaped using standard techniques.

**PRECAUTION:** When shaping the guidewire tip, do not use an object with a sharp edge as this may damage the guidewire.

#### DEPLOYMENT PROCEDURE

Deploy the Spectre guidewire according to the following steps:

##### Deployment Using Bare Wire Technique

1. Following placement of a guide catheter into the vasculature, insert a guidewire introducer through the hemostasis valve attached to the guide catheter. Insert the distal tip of the guidewire through the introducer and into the guide catheter.
2. Under fluoroscopy, advance the guidewire out of the guide catheter and into the vasculature. Attach a torque device to the proximal end of the guidewire if desired.
3. Once the guidewire is in the desired location within the vasculature, secure the guidewire and remove the guidewire introducer and torque device, if applicable.
4. Backload an interventional device over the guidewire and into the vasculature.

##### Deployment Using Preload Technique

1. Insert the guidewire into the guidewire lumen hub of the interventional device.  
**NOTE:** If using a rapid-exchange interventional device, insert the proximal end of the guidewire into the distal tip of the interventional device.
2. Advance the guidewire until the guidewire tip is flush with the tip of the interventional device.
3. Following placement of a guide catheter into the vasculature, insert the distal tip of the guidewire and interventional device assembly into the hemostasis valve attached to the guide catheter.
4. Under fluoroscopy, advance the guidewire and interventional device assembly through the guide catheter until it is just proximal to the tip of the guide catheter. Attach a torque device to the proximal end of the guidewire if desired.
5. Under fluoroscopy, continue advancing the guidewire until it is in the desired location within the vasculature.
6. Secure the guidewire and advance the interventional device over the guidewire.

##### Guidewire Extension (200cm only)

1. To facilitate interventional device exchange over the 200cm guidewire while maintaining guidewire position, first secure the guidewire.
2. Connect the proximal end of the guidewire to the Abbott Vascular DOC Guide Wire Extension using the instructions included with the extension.
3. Maintain guidewire position while withdrawing the interventional device over the guidewire and exchanging it for a new interventional device.

##### Guidewire Removal

1. Once the intervention is complete, carefully withdraw the guidewire from the vasculature.  
**NOTE:** If guidewire exchange is desired, first remove the original guidewire from the vasculature and then deploy the new guidewire using the Bare Wire Technique or Preload Technique described above.
2. Dispose of the guidewire following standard hospital procedures.

#### STORAGE & HANDLING

No special storage or handling conditions.

**Teleflex™**

Teleflex Medical LLC  
 3015 Carrington Mill Blvd  
 Morrisville, NC 27560 USA  
 USA: (866) 246 6990  
 International: +1 919 544 8000

## LIMITED WARRANTY

Teleflex Medical LLC warrants that the Spectre guidewire is free from defects in workmanship and materials prior to the stated expiration date. Liability under this warranty is limited to refund or replacement of any product, which has been found by Teleflex Medical LLC to be defective in workmanship or materials. Teleflex Medical LLC shall not be liable for any incidental, special or consequential damages arising from the use of the Spectre guidewire. Damage to the product through misuse, alteration, improper storage or improper handling shall void this limited warranty.

No employee, agent or distributor of Teleflex Medical LLC has any authority to alter or amend this limited warranty in any respect. Any purported alteration or amendment shall not be enforceable against Teleflex Medical LLC.

THIS WARRANTY IS EXPRESSLY IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER OBLIGATION OF TELEFLEX MEDICAL LLC.

## PATENTS AND TRADEMARKS

May be covered by one or more U.S. or international patents.

See: [www.teleflex.com/patents-intv](http://www.teleflex.com/patents-intv)

Spectre is a trademark of Teleflex Innovations S.à.r.l., or Teleflex Medical, each a part of Teleflex Incorporated.

DOC is a registered trademark of Abbott Cardiovascular Systems, Inc.

See the International Symbols Glossary on page 7.

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DOC es una marca registrada de Abbott Cardiovascular Systems, Inc.